Geistlich Bio-Oss Collagen®

Bone Grafting Material

Geistlich Bio-Oss Collagen® is a sterile, biocompatible bone mineral plus collagen for bone grafting in periodontal, oral and maxillofacial surgery.

COMPOSITION:

Geistlich Bio-Oss Collagen® is a mixture of spongiosa (cancellous) bone granulate and 10% porcine collagen fibers in a block form, and is sterilized by γ -irradiation.

PROPERTIES / ACTION:

The anorganic bone matrix of Geistlich Bio-Oss Collagen® has physical and chemical properties comparable to the mineralized matrix of human bone. The collagen facilitates handling of the graft particles and acts to hold the Geistlich Bio-Oss® at the desired place. The consistency of this material readily allows it to take the shape of the defect. The graft is slowly resorbed and replaced by new bone.

INDICATIONS FOR USE:

Geistlich Bio-Oss Collagen® is intended for the following uses:

- Augmentation or reconstructive treatment of the alveolar ridge.
- Filling of periodontal defects.
- Filling of defects after root resection, apicoectomy, and cystectomy.
- Filling of extraction sockets to enhance preservation of the alveolar ridge.
- Elevation of the maxillary sinus floor.
- Filling of periodontal defects in conjunction with products intended for Guided Tissue Regeneration (GTR) and Guided Bone Regeneration (GBR).
- Filling of peri-implant defects in conjunction with products intended for Guided Bone Regeneration (GBR).

CONTRAINDICATIONS:

Geistlich Bio-Oss Collagen® should not be used in the presence of infected wounds.
Geistlich Bio-Oss Collagen® should not be used in patients with a known allergy to collagen.

DIRECTIONS FOR USE:

The general principles of sterile handling and patient medication must be followed when using Geistlich Bio-Oss Collagen®.

- The filling of periodontal defects with Geistlich Bio-Oss Collagen® requires successful treatment of the periodontal lesion (e.g., root planing, debridement of granulation tissue) prior to implantation. The defect should be covered with a membrane (e.g., Geistlich Bio-Gide®) for optimal tissue regeneration.
- After exposure of the bony defect with mucoperiosteal flap, all granulation tissue must be carefully removed.
- The material should be placed in direct contact with well vascularized, bleeding bone surfaces.
- Cortical bone should be mechanically perforated to facilitate ingrowth of new blood vessels and bone forming cells.
- Application: Geistlich Bio-Oss Collagen® may be cut to the required size with sterile tweezers and scissors, in dry form or after moistening with blood or physiological saline solution. The material is applied using sterile instruments (e.g., tweezers).
- If necessary, the material can be slightly molded in situ using a spatula or similar instrument.
- Overfilling of the defect should be avoided.
- It is advisable to cover Geistlich Bio-Oss Collagen® with a membrane barrier (e.g., Geistlich Bio-Gide®).
- When closing the wound, the soft tissue flap should completely cover the implanted Geistlich Bio-Oss Collagen®, and should be fixed by sutures.
- If primary wound closure cannot be achieved completely, further mobilization of the flap (incision through the periosteum) should be performed.
- A surgical dressing may be placed over the wound for one to two weeks.
- For improved bone formation in large osseous defects with one or two bony walls, Geistlich Bio-Oss Collagen® may be mixed with autogenous particulate spongiosa (cancellous) bone and marrow or covered with a cell occlusive membrane (e.g., Geistlich Bio-Gide®).

PRECAUTIONS:

Geistlich Bio-Oss Collagen® should only be used by trained dentists or oral surgeons.

In order to facilitate the formation of new bone, Geistlich Bio-Oss Collagen® should only be implanted in direct contact with well vascularized bony tissue (selective osteoplasty of adjacent cortical bone may be necessary) and if necessary microfracturation of the adjacent bone might be performed.

From experience, mechanical loading (compression loading) of Geistlich Bio-Oss augmented areas is possible after 6 months at the earliest. Importantly the appropriate timing of implant insertion depends on the local bone volume at the time of implant insertion.

In larger defects a mixture of autogenous bone or bone marrow may improve the formation of new bone.

Geistlich Bio-Oss Collagen® should be used with special caution in patients with:

- Acute or chronic infection (osteomyelitis) at the surgical site
- Metabolic diseases (diabetes, hyperparathyroidism, osteomalacia)
- Osteoporosis
- Severe renal dysfunction, severe liver disease
- High dose corticosteroid therapy
- Vascular impairment at the implant site
- Autoimmune disease
- Radiotherapy
- Heavy smoking

Effect on pregnant or lactating women is not known.

ADVERSE REACTIONS:

Incompatibility reactions with Geistlich Bio-Oss Collagen® cannot be excluded. As Geistlich Bio-Oss Collagen® contains collagen, in very rare cases allergic reactions may occur. Possible complications which may occur with any surgery include swelling at the surgical site, flap sloughing, bleeding, local inflammation, bone loss, infection or pain.

STORAGE AND HANDLING:

If the sterile packaging is damaged or opened the product must not be used.

The contents of the blister package are designed for single use only. Discard any unused material after opening.

Resterilization is not possible.

Do not use after the expiration date.

Store at controlled room temperature 15°-25°C (59°-77°F) and in a dry place.

PRESENTATION: Geistlich Bio-Oss Collagen®:

Re-Order #:	Packaged:
20141	1 block 100 mg Geistlich Bio-Oss® spongiosa (cancellous) granules and 10% collagen
20142	1 block 250 mg Geistlich Bio-Oss® spongiosa (cancellous) granules and 10% collagen
20143	1 block 500 mg Geistlich Bio-Oss® spongiosa (cancellous) granules and 10% collagen

Geistlich Combi-Kit Collagen®

Re-Order #:	Packaged:
20146	1 block 100 mg Geistlich Bio-Oss® spongiosa (cancellous) granules and 10% collagen, and 1 Geistlich Bio-Gide® membrane, 16 x 22 mm

Geistlich Perio-System Combi-Pack

Re-Order #:	Packaged:
20147	1 block 100 mg Geistlich Bio-Oss® spongiosa (cancellous) granules and 10% collagen, and 1 Geistlich Bio-Gide® Perio membrane, 16 x 22 mm

Distributed by:

Geistlich Pharma North America Inc.
Princeton, New Jersey 08540
1-(855) 799-5500
www.geistlich-na.com

Manufacturer:

Geistlich Pharma AG
CH-6110 Wolhusen
Switzerland

Made in Switzerland

CAUTION: Federal law restricts this device to sale by or on the order of a dentist or physician.
package insert: www.geistlich-na.com

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